

MIDWEST Caries»I.D.™

PROBE DIRECTIONS FOR USE

Indications

The Midwest Caries I.D.™ Probe is to be used only with the Midwest Caries I.D. Detection Handpiece to aid in the detection of caries in pits and fissures and interproximal areas on posterior teeth that have not been restored.

Contraindications

The Midwest Caries I.D. Probe is not for use with any detection device other than the Midwest Caries I.D. Detection Handpiece.

Connecting The Probe

Note: Probe is not shipped sterile, autoclave prior to installation per Infection Control Procedures found in the Midwest Caries I.D. Detection Handpiece Directions for Use.

1. Ensure that the device is "OFF", the inside of the Probe is dry, and that the alignment notch and tab are aligned. Holding the Detection Module and Probe as shown in illustration, insert the Detection Module into the Probe. Gently seat the Probe's fiber connections into the mating holes in the Detection Module before fully seating the Probe. Do not turn or twist the Probe during assembly.
2. After assembling, gently wipe the tip of the Probe with gauze dampened with isopropyl alcohol. This eliminates the presence of residues. *Note: Gauze not included in package.*

Removing The Probe

1. Firmly hold the lower portion of the shell with the Probe tip facing the ground.
2. Place your hand palm up on the Probe.
3. Remove the Probe by pulling in a straight line: Do not try to unscrew or twist the Probe.



Illustrations of proper hand positioning when connecting or remove the Probe.

For all other instructions, including Warnings, Precautions, Operating Procedures, Infection Control Procedures, Probe Polishing, Fiber Optic Cleaning and Troubleshooting, refer to your Midwest Caries I.D. Detection Handpiece Directions for Use.

DENSPLY
PROFESSIONAL

1301 Smile Way | York, PA 17404 | 800-989-8826 | www.professional.dentsply.com